



OPEN

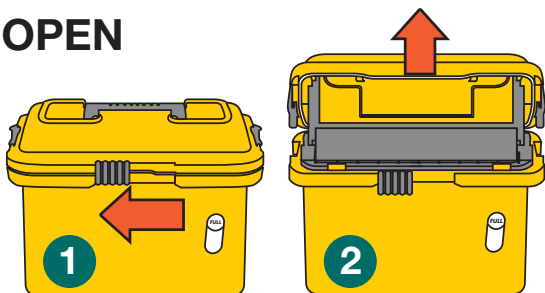

1. Slide front lock to left.

2. Lift lid until locked open.


SYRINGES

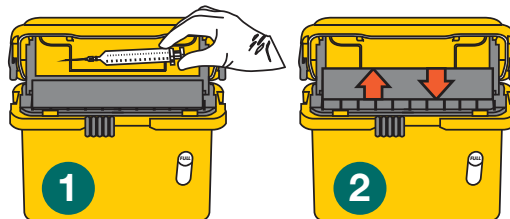

1. Drop syringe horizontally onto tray.

2. Tray will empty itself.


LIGHT SHARPS

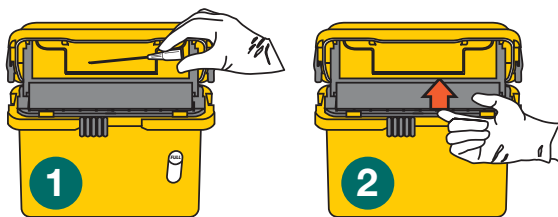

1. Drop sharp horizontally onto tray.


2. Manually lift tray to empty.

FULL

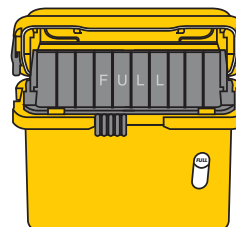

When tray remains vertical, lock and replace collector.


CLOSE

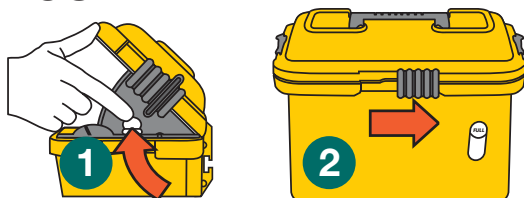

1. Press both side supports inward, close lid.

2. Slide front lock fully to right.

LOCK

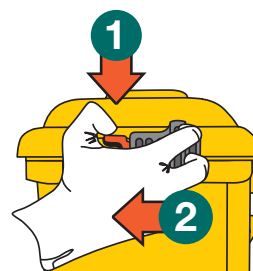

1. Depress red tabs.

2. Slide locks completely forward.

NOTE: Side locks are permanent!

REMEMBER:

- ALWAYS ENSURE SHARP HAS EMPTIED FROM TRAY • NEVER PUT HANDS BEYOND TRAY LIP
- ALWAYS CLOSE COLLECTOR BEFORE MOVING/REMOVING COLLECTOR
- DO NOT DEFACE OR WRITE ON COLLECTOR
- ALWAYS DISPOSE OF SHARPS AND SHARPS SAFETY DEVICES INTO SHARPS COLLECTORS