



SharpSmart Access Plus

INSTRUCTIONS FOR USE


OPEN

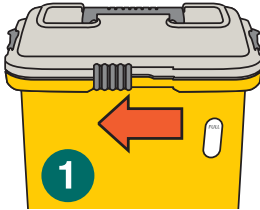

1. Slide front lock to left.

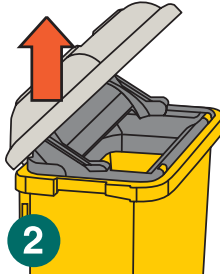


2. Lift lid to 45 degrees for semi-access opening.

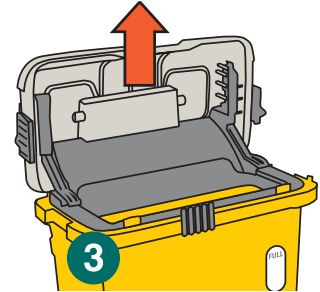


3. Lift lid to 90 degrees for full access opening.


USAGE

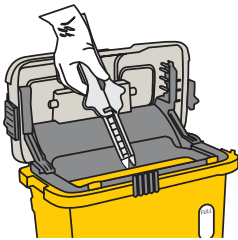

1. Drop items point downwards into opening.

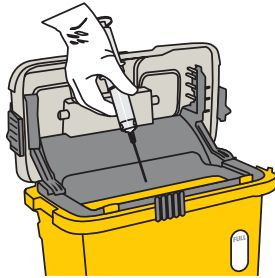


FULL

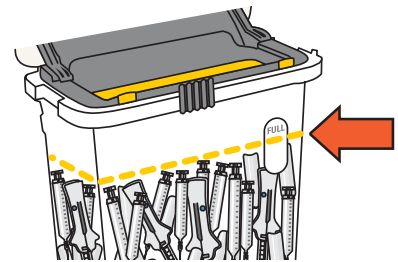

When sharps reach fill level, Access Plus collector is full and must be locked and replaced.


CLOSE

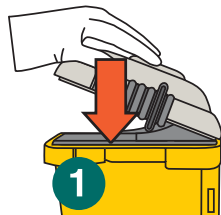

1. Press lid down firmly.
2. Slide front lock fully to right.



LOCK

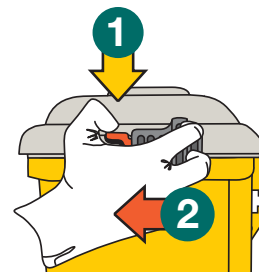

1. Depress red tabs.
2. Slide locks completely forward.
NOTE: Side locks are permanent!

REMEMBER:

- NEVER PUT HANDS INTO SHARPS COLLECTOR
- ALWAYS CLOSE COLLECTOR BEFORE MOVING/REMOVING COLLECTOR
- DO NOT DEFACE OR WRITE ON COLLECTOR
- ALWAYS DISPOSE OF SHARPS AND SHARPS SAFETY DEVICES INTO SHARPS COLLECTORS