



# SharpSmart Access Plus

## INSTRUCTIONS FOR USE


### OPEN

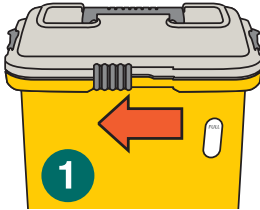

1. Slide front lock to left.

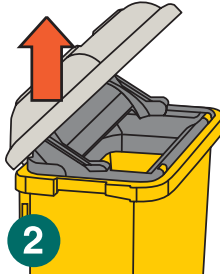


2. Lift lid to 45 degrees for semi-access opening.

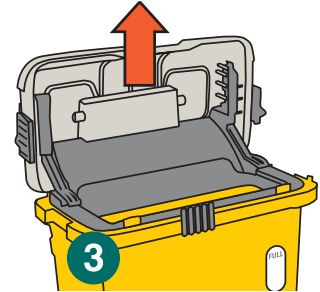


3. Lift lid to 90 degrees for full access opening.

### USAGE

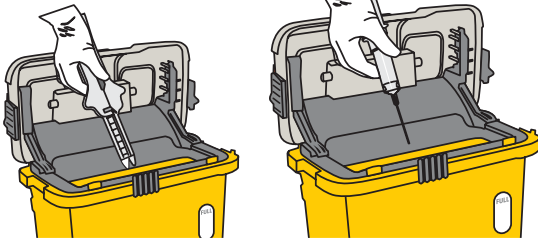


1. Drop items point downwards into opening.

### FULL

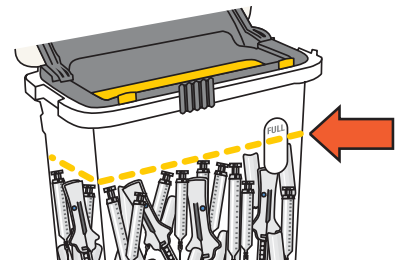

When sharps reach fill level, Access Plus collector is full and must be locked and replaced.

### CLOSE

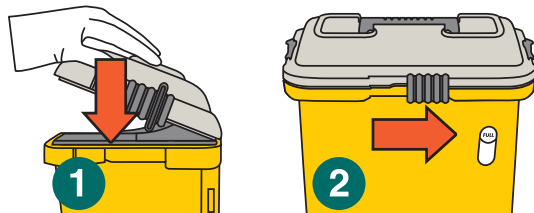

1. Press lid down firmly.  
2. Slide front lock fully to right.

### LOCK



2. Slide locks completely forward.  
**NOTE: Side locks are permanent!**

### REMEMBER:

- NEVER PUT HANDS INTO SHARPS COLLECTOR
- ALWAYS CLOSE COLLECTOR BEFORE MOVING/REMOVING COLLECTOR
- DO NOT DEFACE OR WRITE ON COLLECTOR
- ALWAYS DISPOSE OF SHARPS AND SHARPS SAFETY DEVICES INTO SHARPS COLLECTORS